## **Broadstairs Toilet Frame**

**VR202**

## **Fixing and Maintenance Instructions**

## INTRODUCTION

Thank you for deciding to purchase the Broadstairs Toilet Frame from Aidapt. The Broadstairs Toilet Frame is manufactured from the finest materials and components available, when used correctly it is designed to give many years of reliable trouble-free service.

**Please Note: Always consult with the prescriber/Physiotherapist if you are in any doubt that this product is suitable for your use.**

**NB. This equipment must be installed by a competent person.**

190 kg (30 st.) weight limit.

**Do not exceed the weight limit stated – doing so could put the user at risk.**

## BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.

## CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your toilet frame using a non-abrasive cleaner or mild detergent with a soft cloth and avoid using lime scale remover.


Under no circumstances use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for ABS, Vinyl, Polymer Plastics and Thermo plastic rubbers and Polyurethane Rubber.

**NB. Regular daily cleaning will reduce the build up of limescale on your product.**

Body spillages should be cleaned as and when they occur (giving particular attention to the area where the frame width is extended), as this will avoid causing rust or corrosion which could damage/weaken the frame.

## ADJUSTMENTS TO HEIGHT/WIDTH EXTENSION LEGS

- Remove the 'E' clip from the extension leg and extend all four legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)
- Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see fig. 2).
- Never install the extension legs unevenly or slanting, this will compromise the safety of the user.
- Repeat the process for the width adjustment bars.

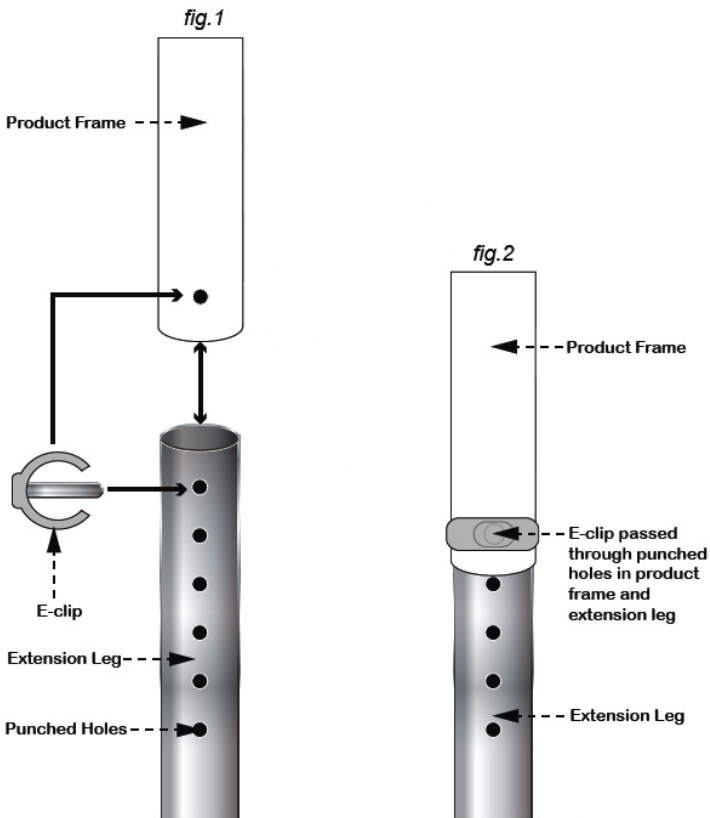## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Proper height adjustment
- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.
- Also check all upholstery for security, splits, etc.
- Width and leg extensions

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns. Please pay particular attention to the width and leg extensions.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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**Altai Germany (Altai Europe Ltd.)