

Angelcare®

FOR YOUR PEACE OF MIND

OWNER'S MANUAL

SIMPLICITY MOVEMENT SENSOR
WITH SOUND BABY MONITOR

Model AC601



DON'T
FORGET

IMPORTANT ADVICE FOR INITIAL SETUP



Movement Alarm

Your monitor comes with the movement alarm activated. When switching on your monitor it will alarm, with or without the Sensor Pad plugged in, if no movement is detected. Deactivate the movement function to use the monitor without the Sensor Pad.



Channel Selection

When monitoring sound and/or movement, the channel options on both Parent and Nursery Unit must be set to the same position: A, B or C. This can be done by sliding the switches on the left side of both units to the markings indicating A, B or C.



Cord Safety

Ensure the Nursery Unit is at least 1 metre away from your baby's cot. The Sensor Pad cord must always be kept out of reach of baby and secured using the cord cover kit provided.



Voice-activated Sound Transmission


Your monitor will transmit when it detects sound from your baby or in their room. You can adjust the microphone sensitivity to suit your needs.

PLEASE FOLLOW THE INSTRUCTIONS CAREFULLY FOR CORRECT SETUP

Please **DO NOT** return this product to the store. For assistance please contact our helpline on **0345 009 1789** or email helpline@babyhelpline.co.uk
Mon - Fri 10am - 4pm, excluding bank holidays.

TABLE OF CONTENTS

.....

 WARNINGS	3
AC601 SETUP REQUIREMENTS	7
STEP 1 - Charging the Parent Unit (15 hours)	8
STEP 2 - Installing the Sensor Pad	10
STEP 3 - Setting up the Nursery Unit.....	13
STEP 4 - Turning on your monitor.....	14
STEP 5 - Testing and getting to know your monitor.....	15
Troubleshooting	18
Limited warranty.....	19
Technical specifications.....	20

OWNER'S MANUAL

KEEP YOUR OWNER'S MANUAL FOR FUTURE REFERENCE.

PLEASE READ THE WARNINGS AND INSTRUCTIONS CAREFULLY BEFORE ASSEMBLING & USING THE PRODUCT.

FAILURE TO FOLLOW THE WARNINGS AND THE INSTRUCTIONS COULD RESULT IN SERIOUS INJURY OR DEATH OR DAMAGES TO THE PRODUCT.

THE SENSOR PAD FEATURE IS NOT FOR USE WITH ADULTS, CHILDREN OR INFANTS OLDER THAN 24 MONTHS.

THE SENSOR PAD IS NOT INTENDED TO BE USED TO DETERMINE IF A CHILD IS OUT OF THE CRIB.

WARNING

Adult assembly required.

Keep small parts away from children when assembling.



This Angelcare Monitor is a Personal Care Product. Please DO NOT return this to the store where you purchased it from. If you have any questions or require assistance please call the Angelcare Helpline on 0345 009 1789 or email helpline@babyhelpline.co.uk.

The Baby Helpline is open Monday to Friday, 10am to 4pm. The helpline is closed on bank holidays.


For additional FAQs and to download instruction manuals visit; www.angelcare.co.uk.

To order spare parts for your Angelcare Baby Monitor visit; www.megababy.co.uk.

⚠ WARNINGS

1. THIS PRODUCT CANNOT REPLACE RESPONSIBLE ADULT SUPERVISION. This product is designed to aid in the monitoring of your child. When using the monitor, you must still provide the supervision necessary for the continued safety of your child. Monitoring of premature babies, or those considered to be at risk, should always be carried out under the supervision of a health professional. This monitor **SHOULD NOT** be used as a medical device, nor as a device to prevent Sudden Infant Death Syndrome (SIDS).

2. WARNING Strangulation Hazard — Children have STRANGLED in cords.

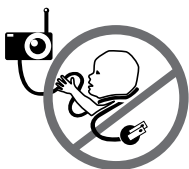

Keep adapter cords out of the reach of children (more than 1 m away). **NEVER** use Sensor Pad without the protective cord covering. Check protective cord covering before each

use. Immediately discontinue use of Sensor Pad if protective covering is damaged or broken.

3. Never use extension cords with AC adapters. **ONLY** use the AC adapters provided.

4. When an AC Adapter is plugged into an electrical outlet, **DO NOT** touch the exposed end.

5. **ALWAYS** be sure that both the transmitter and receiver are working properly and are within range of each other. Test monitor before first use, periodically and when changing location of the Nursery Unit.

6. **ALWAYS** be sure to immediately check on baby whenever an alarm sounds.

7. **DO NOT** use monitor near water (such as bath tub, sink, etc.).



8. Keep monitor **AWAY** from heat sources (such as stoves, radiators, etc.).

9. **ALWAYS** place both the Nursery Unit and Parent Unit on a flat surface in an upright position and allow for proper ventilation. **DO NOT** place on sofas, cushions, beds, etc. which may block ventilation.

10. This monitor uses public airwaves to transmit signals. The Parent Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned **OFF** when not in use. If you experience interference, change the channel on both units.

11. This product is **NOT** a toy. **DO NOT** allow children to play with it.

12. This product contains small parts. Adult assembly required. Exercise care when unpacking and assembling the product and keep small parts away from children.

13. When using the monitor, the room temperature should be between 10 °C and 40 °C.

⚠ BATTERY WARNING

14. Use **ONLY** AAA RECHARGEABLE batteries for the Parent Unit. **DO NOT** mix old and new batteries. **DO NOT** mix alkaline, standard or rechargeable batteries. Insert batteries with correct polarity. **DO NOT** short-circuit supply terminals. **ONLY** use the recommended batteries or equivalent (volts and size) and the correct adapter. Remove batteries when product is stored for long periods of time or when batteries are exhausted. Dispose exhausted batteries properly. **DO NOT** recharge non-rechargeable batteries. Rechargeable batteries are **ONLY** to be charged under adult supervision. Keep all batteries away from children.

15. IMPORTANT SAFETY INSTRUCTIONS

15.1 Read these instructions - All the safety instructions should be read before the product is operated.

15.2 Keep these instructions. The safety and operating instructions should be retained for future reference.

15.3 Heed all warnings - All warnings on the product and in the operating instructions should be adhered to.

15.4 Follow all instructions - All operating and use instructions should be followed.

15.5 Water and moisture - DO NOT use this apparatus near water. For example, near a bath tub, wash bowl, kitchen sink, laundry tub, in a wet environment or near a swimming pool and the like. Care should be taken so that objects DO NOT fall onto units and that liquids are not spilled into either unit or Sensor Pad through its openings.

15.6 Cleaning - Disconnect all units before cleaning. DO NOT immerse in water any part of the monitor. Clean only with dry cloth.

15.7 Ventilation - DO NOT block any ventilation openings. Allow for proper ventilation when units are in use. DO NOT cover the Nursery Unit or Parent Unit with any object such as a blanket. DO NOT place it in a drawer or in any location which would muffle the sound or interfere with the normal flow of air.

15.8 Heat sources - DO NOT install near any heat sources such as radiators, heat registers, stoves, TVs or other apparatus (including amplifiers) that produce heat. Heat can damage the case or electrical parts.

15.9 Power cord protection - Protect the Sensor Pad cord and Adapter's power cord from being walked on or pinched by furniture or other items - Power supply cords should be routed so that they are not likely to be walked on or pinched by items upon or against them, paying particular attention to cords and plugs, convenience receptacle, and the point where they exit from the product.

15.10 Accessories - Only use attachments/accessories specified by the manufacturer. DO NOT place this product on an unstable cart, stand, tripod, bracket or table. The product may fall, causing serious injury to a child or adult and serious damage to the product. Use ONLY with a cart, stand, tripod, bracket or table recommended by the manufacturer or sold with the product. Any mounting of the product should follow the manufacturer's instructions.

15.11 Unplug this appliance during lightning storms or when not used for long periods of time.

15.12 Servicing - DO NOT attempt to service this product yourself as opening or removing covers (except battery doors) may expose you to dangerous voltage or other hazards.

15.13 Replacement parts - When replacement parts are required, be sure the replacement parts specified by the manufacturer have the same characteristic as the original part. Unauthorized substitution may result in fire, electric shock or other hazards.

15.14 Usage - This baby monitor is intended as an aid. It is NOT a substitute for responsible and proper adult supervision and should not be used as such. Use this appliance ONLY for its intended use as described in this manual.

15.15 Power Sources - Operate this product only from the type of power source indicated on the type plate of the adaptor. If you are unsure of the type of power supply to your home, consult your dealer or local power company.

15.16 Overloading - DO NOT overload wall outlets, extension cords or integral convenience receptacles as this can result in a risk of fire or electric shock.

15.17 Object and Liquid Entry - NEVER push objects of any kind into this product through casing slots as they may touch dangerous voltage points or short out parts that could result in a risk of fire or electric shock. NEVER spill liquid of any kind on the product.

15.18 Antenna - The antenna used for this transmitter must be installed to provide a separation distance of at least 20 cm from all persons and must not be collocated or operating in conjunction with any other antenna or transmitter.

15.19 Power Lines - An outside antenna system should NOT be located in the vicinity of overhead power lines or other electric light or power circuit, or where it can fall into such power lines or circuits as contact with them may be fatal.

15.20 WARNING Strangulation Hazard – Children have **STRANGLING** in cords. Keep adapter cords out of the reach of children (more than 1 m away) NEVER use Sensor Pad without the protective cord covering. Check protective cord covering before each use. Immediately discontinue use of

Sensor Pad if protective covering is damaged or broken.

15.21 Interference - This monitor uses public airwaves to transmit signals. The Parent Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned OFF when not in use. If you experience interference, change the channel on both units.

15.22 Vibration Source - When the monitor is in use, DO NOT use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement other than your baby's. Be sure all vibration sources are eliminated before using the monitor.

You may experience false alarms for several reasons. The most common reason is the Nursery Unit has not been turned OFF after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly.

Be sure the Sensor Pad is on a flat, firm surface with the printed side facing up. The sensitivity level may also need to be adjusted. Refer to step 5.3. Always be sure to immediately check on baby whenever an alarm sounds.

15.23 Sensor Pad - If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 33 x 33 cm and 6 mm thick. Only use your Angelcare® monitor in a stationary

sleeping environment such as a crib where the Sensor Pad can rest on a completely flat, firm surface. The Sensor Pad is made of plastic and does not permit the air to circulate. Therefore, mildew build up could develop underneath the mattress in the sensor pad area. We suggest rotating the mattress regularly to prevent any mildew.

16. IMPORTANT SAFETY INSTRUCTIONS RELATED TO USE OF BATTERIES

16.1 Use ONLY the type and size of batteries specified in the owner's manual. There is a risk of explosion if batteries are replaced with an incorrect type.

16.2 Use ONLY AAA rechargeable batteries and/or the AC Adapters provided for the Parent Unit. You must use only rechargeable batteries in the Parent Unit. DO NOT use alkaline batteries or you will severely damage the unit.

16.3 Charge the batteries for 15 hours prior to first use. Be sure to have the monitor in the OFF position during this period. ONLY use the AC Adapters included with your monitor.

16.4 DO NOT use these adapters with any other equipments.


16.5 When unplugging the AC Adapter from the Nursery Unit, be sure it is switched to the OFF position. If it isn't, the unit will revert to battery mode if batteries are installed.

16.6 DO NOT use rechargeable batteries in the Nursery Unit. Use alkaline batteries ONLY.

16.7 DO NOT dispose batteries in fire. They may explode. Check with local regulations for possible special disposal instructions.

16.8 Exercise care in handling batteries in order not to short out the batteries with conducting materials such as rings, bracelets and keys. Overcharging, short circuiting, reverse charging, mutilation, or incineration of batteries must be avoided to prevent one or more of the following occurrences: release of toxic materials, release of hydrogen and/or oxygen gas, rise in surface temperature.

16.9 DO NOT attempt to recharge batteries provided with or identified for use with this product that are not intended to be charged. The batteries may leak corrosive electrolyte or explode.

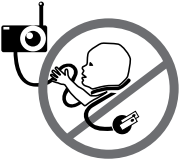

16.10 DO NOT attempt to rejuvenate the batteries provided with or identified for use with this product by heating them. Sudden release of battery electrolyte may occur, causing burns or irritation to eyes or skin.

16.11 If and when necessary, the batteries must be replaced with identical new ones or equivalent (volts and size). If a battery to be replaced is connected with other batteries in series, replace the other batteries at the same time as well. DO NOT mix old and new batteries.

16.12 Reverse polarity installation of the batteries in the end product must be avoided. Reverse insertion of batteries can cause charging, and that may result in leakage or explosion.

16.13 Remove batteries from this product if you store the product over 30 days because the batteries could leak and damage the product.

16.14 Discard 'dead' batteries as soon as possible since 'dead' batteries are more likely to leak in a product. If a battery has leaked or vented, it should be replaced immediately using protective gloves.

16.15 DO NOT store this product, or the batteries provided with or identified for use with this product, in high-temperature areas.

FC WARNING

(North America Only)

17.1 Modifications not authorized by the manufacturer may void users authority to operate this device.

17.2 NOTE: This equipment has been tested and found to comply with Part 15 of the FCC Rules and the Industry Canada licence-exempt RSS standard. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

17.3 However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment OFF and ON, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

17.4 This device complies with part 15 of the FCC Rules and the Industry Canada licence-exempt RSS standard. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and

(2) This device must accept any interference received, including interference that may cause undesired operation.

FCC AND IC REGULATIONS

RF radiation exposure statement

The Nursery Unit complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. **Caution:** To maintain the compliance with the FCC's RF exposure guideline, install the Nursery Unit at least 20 cm from ALL persons. The transmitter of this device must not be co-located or operating in conjunction with any other antenna or transmitter.

Industry Canada

This product meets the applicable Industry Canada technical specifications.
CAN ICES-3 (B)/NMB-3 (B)

SETUP REQUIREMENTS


MOVEMENT SENSOR
WITH SOUND MONITOR

Model AC601


INCLUDED:




Parent Unit

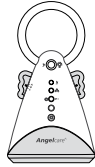


Nursery Unit

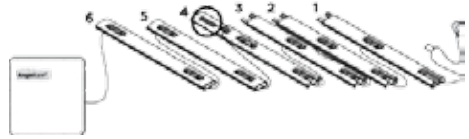

1 Sensor Pad (AC601) with pre-installed cord covers

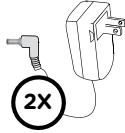

2 AC Adapters

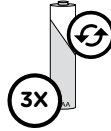

3 Rechargeable AAA Batteries
for Parent Unit

YOU MUST PREPARE:

One hardboard (not included)
Thickness: **6 mm**
Minimum dimensions:
33 cm x 33 cm
Maximum dimensions:
equal to the mattress size




Hardboard

IMPORTANT

The hardboard must be larger than the Sensor Pad surface area.

WHAT YOU WILL NEED (NOT INCLUDED):

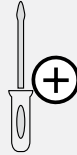

Miniature Phillips
Screwdriver




4 Alkaline
AAA Batteries
for Nursery Unit

BABY'S ROOM:


IMPORTANT

Place the crib near a supporting wall and far from air currents. Install the Nursery Unit on a flat, stable surface facing the crib at a distance of at least **1 meter**.

Make sure to plug the adapter to the socket outlet nearest to the equipment and that it is easily accessible.

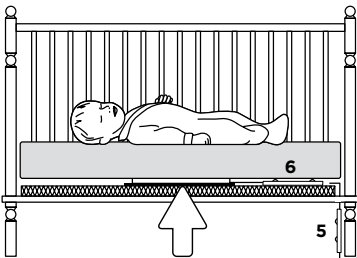


Sensor Pad

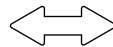


IMPORTANT

Keep
at least
1 m
apart


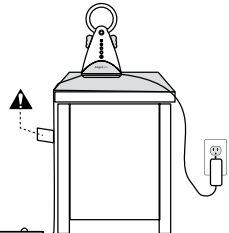


STEP 1 - CHARGING THE PARENT UNIT (15 HOURS)

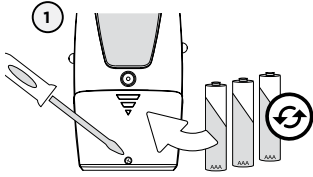

IMPORTANT

Never use regular alkaline batteries in the Parent Unit; they may explode.

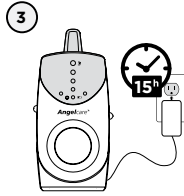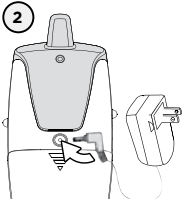

- 1) Remove the battery cover and insert the Rechargeable Batteries (included).
- 2) Connect the AC Adapter to the Parent Unit and into an electrical outlet.
- 3) Charge the unit for **15 continuous hours** (prior to first use).

NOTE:

To optimize your battery performance please refer to Battery Guidelines in the Trouble Shooting section.

1.1 - BATTERIES

HOW TO CHECK THE BATTERIES:

Unplug the AC Adapters from both units.

- If the Nursery or the Parent Unit Power Indicator does not light up, replace the batteries.



PARENT UNIT LOW BATTERY ALARM


- Parent Unit will emit a rapid ticking sound.
- Battery Indicator slowly flashes when batteries need to be recharged.

NURSERY UNIT BATTERY ALARM


- Low Battery Indicator flashes when batteries need to be changed.
- If the batteries are not replaced, the monitor will stop working.


HOW TO CHARGE THE BATTERIES IN THE PARENT UNIT:

Plug the AC into the Parent Unit and a working electrical outlet. Your Parent Unit can function with the batteries at 1/2 capacity but it is recommended that they are fully charged before unplugging from the AC Adaptor and power outlet for maximum use.

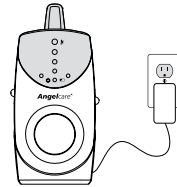


PARENT UNIT:

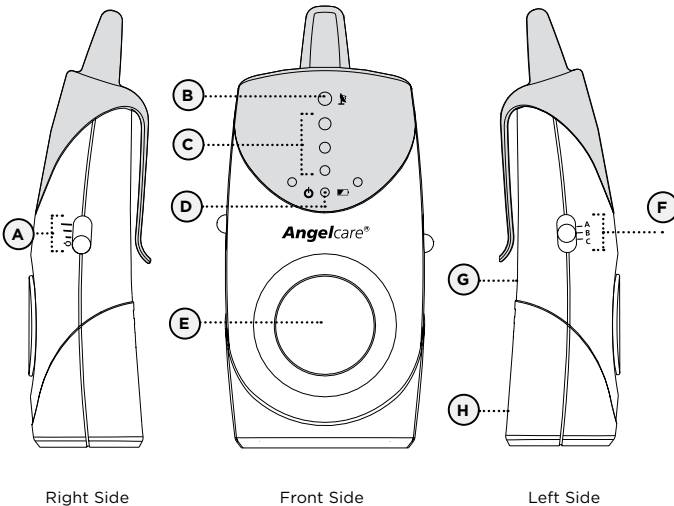

- A) Volume / Power** 
 - High
 - Medium
 - Low
 - OFF
- B) Out-of-Range Indicator**
- C) Bar Graph LEDs**
 - Sound Lights
 - Alarm
 - Charging Indicators
- D) Power Indicator / Low Battery Indicator**
- E) Speaker**
- F) Channel Selection (A, B, C)**
- G) AC Adapter Outlet (not shown)**
- H) Battery Door (not shown)**

STEP 2 - INSTALLING THE SENSOR PAD

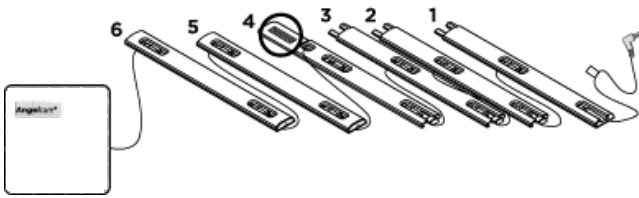


Sensor Pad with pre-installed cord covers

Cord covers 1, 2, 3, 4:
Position on floor

Cord cover 4:
Contains double sided tape to securely install beneath crib leg

Cord cover 5:
Position vertically between mattress support and floor

Cord cover 6:
Place under the mattress

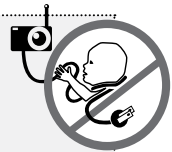


⚠ WARNING
STRANGULATION HAZARD.

Children have STRANGLED in cords.


- Keep adapter cords out of the reach of children (more than 1 meter away).
- Because of the serious strangulation risk to infants and toddlers, parents and caregivers should never place the Nursery Unit within 1 meter of a crib.
- **NEVER** use Sensor Pad without the cord covers. Check cord covers before each use to ensure it is securely placed under the leg of the crib. Discontinue use of Sensor Pad if any cord covers are damaged or broken.
- **NEVER** use extension cords with AC adapters. **ONLY** use the AC adapters provided. This product is not a toy. Never allow children to play with it.

2.1 - SELECT A LOCATION FOR THE NURSERY UNIT


Select a location for the Nursery Unit that is at least **1 m away from the crib.**

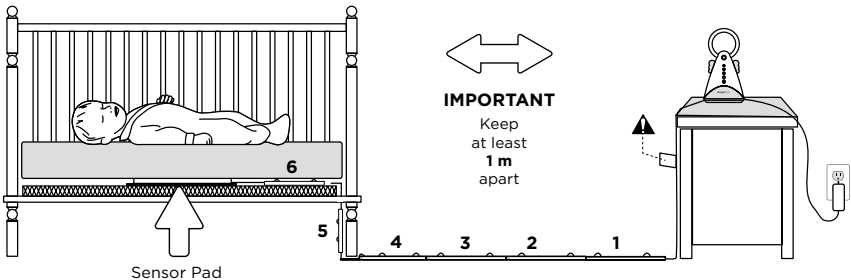

2.2 - SELECT THE CRIB LEG YOU WILL BE USING TO SECURE THE CORD COVERS

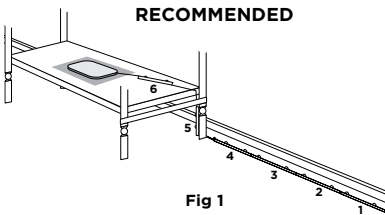


OR




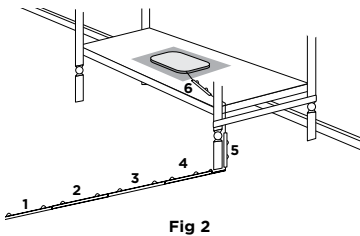


Use the crib leg by the wall closest to the Nursery Unit (**Fig 1**). In cases where the Nursery Unit is across the room, use the leg closest to the Nursery Unit to position the covered wire (**Fig 2**). It is always preferable that the covered cord run along a wall.

2.3 - INSTALL THE SENSOR PAD ON THE MATTRESS SUPPORT

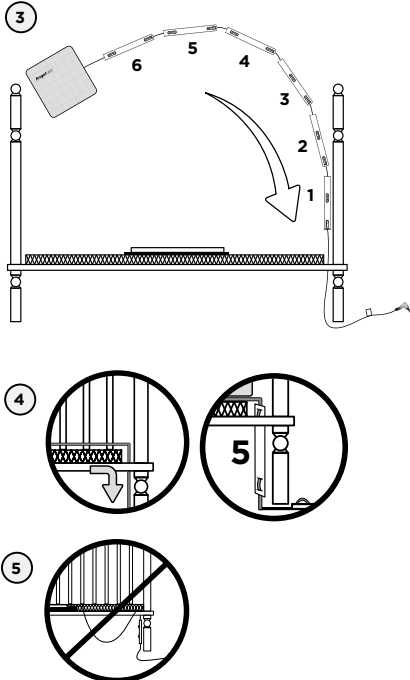

- 1) Remove the mattress from the crib.
- 2) Place the Sensor Pad - printed side up - on top of the **hardboard** at the center of the crib.
- 3) Run the connection pin, warning label and the first 5 cord covers down along the backside's of the crib leg, **between the mattress support and the inside edge of the crib**.
- 4) Bring the first 4 cord covers down the floor, and keep the 5th cord cover vertically, along the backside of the crib leg, (**between the mattress and the inside edge of the crib**).
- 5) Remove any cord slack to make sure no parts of the cord are loose.

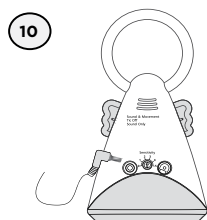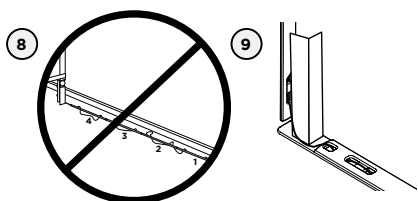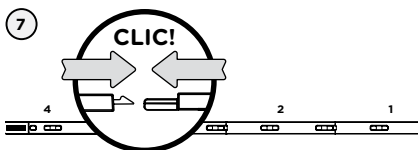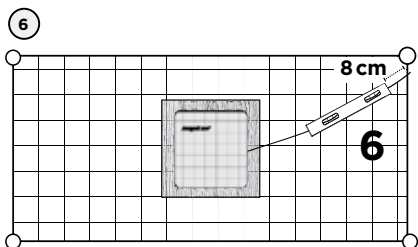

- 6) Place the 6th cord cover approximately 8 cm from the edge of the mattress support.
- 7) Snap together the remaining cord covers that will run along the floor.
- 8) Remove any cord slack to make sure no parts of the cord are loose.
- 9) Peel off the double-sided tape on the 4th cord cover and place it securely under the crib leg.
- 10) Install the Nursery Unit on a flat, stable surface facing the crib at a distance of at least 1 m and plug the Sensor Pad cord into the Nursery Unit.
- 11) Check to make sure the Sensor Pad has not moved from position.
- 12) Place the mattress back inside the crib.

..... The Sensor Pad functions with any size
 and type of mattress except memory-foam
 mattresses and hollow frame mattresses.

..... The hardboard maintains a constant
 pressure on the Sensor Pad, enabling it to
 detect the baby's movements even when
 the baby is at the edges of the crib.

WARNING




STRANGULATION HAZARD


- Remove all slack from Sensor Pad Cord.
- When lowering the mattress support as your child grows, slide the vertical Cord Cover (5) upward so it remains approximately in the center between the mattress support and the floor. Slide the Sensor Pad Cord through the Floor Cover to eliminate any additional slack.

- The Sensor Pad is not designed for use with adults, children or infants older than 24 months.
- Discontinue used of the Sensor Pad when child is older than 24 months. Remove Sensor Pad from crib and store for future use.
- **BEFORE FIRST TIME USE:** To test that the unit is ready for use, refer to instruction manual on how to test that the installation has been done correctly.

STEP 3 - SETTING UP THE NURSERY UNIT

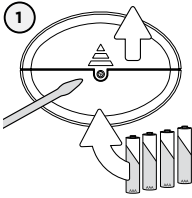


- 1) Remove the battery cover and insert 4 AAA Alkaline Batteries (not included).
- 2) Place the Nursery Unit at a **minimum of 1 m** away from the crib.
- 3) Connect the AC Adapter to the Nursery Unit and plug into an electrical outlet.

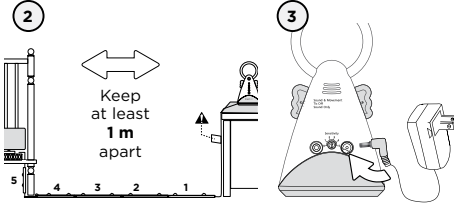


NOTE:

The batteries maintain power supply in case of power outage.

NURSERY UNIT:

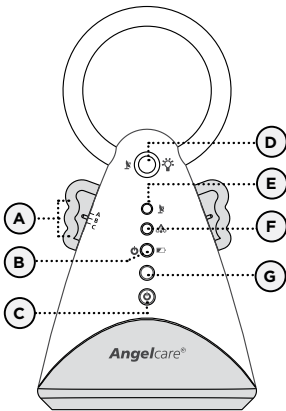


Front Side

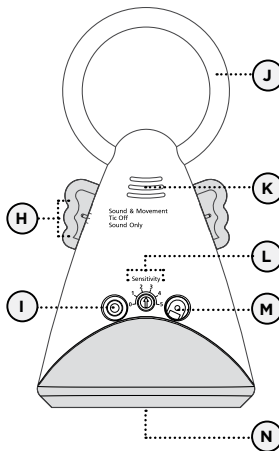

Back Side

- A) Left 'Wing' - Channel Selection (A, B, C)
- B) Power Indicator / Low Battery Indicator
- C) **POWER ON/OFF**
- D) Night Light Button / Out-of-Range Function
- E) Out-of-Range Indicator
- F) Movement Indicator
- G) Microphone
- H) Right 'Wing' - Detection Mode
 - Sound and Movement
 - TIC OFF
 - Sound Only
- I) Sensor Pad Outlet
- J) Night Light
- K) Alarm Sound Speaker
- L) Sensor Pad Sensitivity Dial
- M) AC Adapter Outlet
- N) Battery Door (not shown)

STEP 4 - TURNING ON YOUR MONITOR

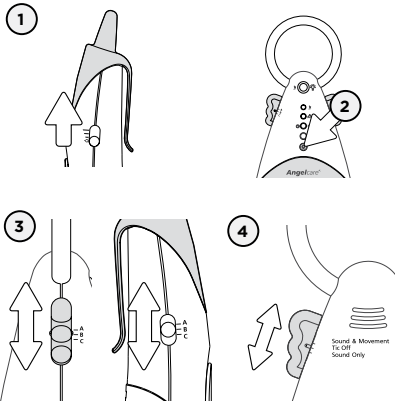

- 1) Slide ON Parent Unit **Volume/Power** button.
- 2) Push Nursery Unit **Power ON/OFF** button for 2 seconds.
- 3) Set both Units on the same channel (A, B or C). Must always be on matching channels.
- 4) Select your detection mode .

Sound and Movement


Allows you to detect baby's sounds and movements as well as hear a TIC sound at each movement.

Tic OFF



Allows you to detect the baby's sounds and movements without the TIC sound.

Sound Only

Allows you to detect ONLY the baby's sounds. Movement detection will be deactivated.

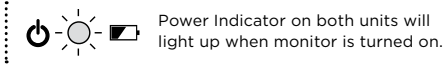

DETECTION MODE FEATURES

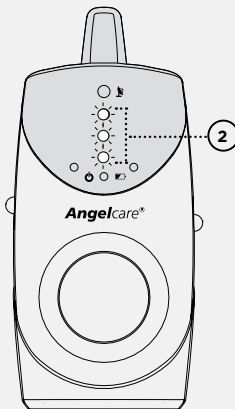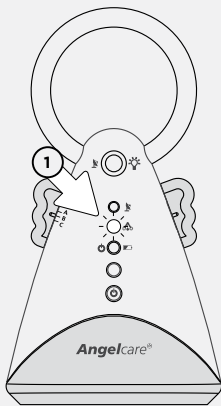

1) Movement Indicator

Light activated by your baby's movement.


2) Sound Lights

Sound lights on the Parent Unit are activated by the sounds of your baby. As the sounds get louder, more lights will illuminate.

STEP 5 - TESTING AND GETTING TO KNOW YOUR MONITOR

5.1 - SENSOR PAD

TO TEST THE SENSOR PAD:

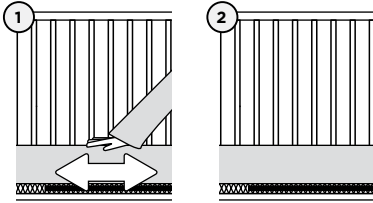

- 1) Move your hand lightly on the mattress.
: The Nursery Unit Movement Indicator will flash continuously.
- 2) Remove your hand from the mattress. After 15 seconds you will hear a 'TIC' pre-alarm sound, and 5 seconds later the Movement Alarm will trigger.
- 3) Repeat by moving your hand lightly in all four corners of the mattress. Ensure the Nursery Unit Movement Indicator is still flashing.


MOTION ALARM

- If NO movement is detected after 15 seconds you will hear a single 'TIC' pre-alarm sound from the Nursery Unit.
- If after a further 5 seconds there is still NO movement detected both the Parent Unit and Nursery Unit speakers will emit a loud warning sound.
- Green light on the Nursery Unit will turn off to indicate NO movement is being detected.

IF THE ALARM DOES NOT SOUND


- Avoid contact with crib while Sensor Pad is in operation.
- Place crib near a supporting wall.
- Adjust the Sensor Pad sensitivity level.

TO STOP THE ALARM

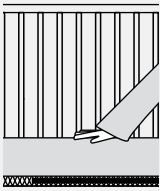

Place your hand back on the mattress.

OR

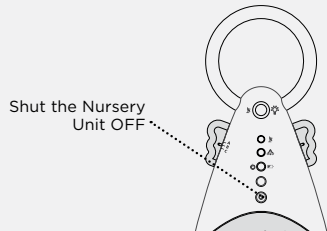


TO ADJUST THE SENSOR PAD SENSITIVITY LEVEL:

Turn the Nursery Unit sensitivity dial from less sensitive (0) to very sensitive (5).

NOTE:

If you are experiencing false alarms, adjust the sensitivity level.

For better result, start by half level increases or decreases.

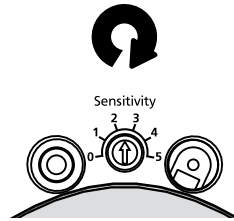

IMPORTANT

- The Sensor Pad is not designed for use with adults, children or infants older than 24 months.
- Once you are finished using the Sensor Pad, please remove it from the baby's crib and store it away.

5.2 - OUT OF RANGE

Your monitor has an Out-of-Range function that advises you when your two units are not communicating. Out-of-Range situations can be caused by distance or by physical interferences.

RANGE:


Open field (Open area without obstructions): up to 250 meters.

TO ACTIVATE THIS FUNCTION:


Press and hold the Nursery Unit Night Light button for 2 seconds.



The Out-of-Range Indicator (orange) on both units will light up to display function's activation.

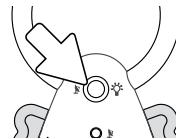


TO STOP THE OUT-OF-RANGE ALARM:

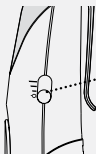

Turn the Parent Unit OFF

OR

Turn the Nursery Unit ON

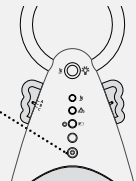

OUT-OF-RANGE ALARM

- Alarm activates when your two units are not communicating.
- The Parent Unit will emit a double beep every 8 seconds.

IMPORTANT

If the Out-of-Range function is activated and you turn **the Nursery Unit OFF** the Out-of-Range Alarm will sound. Turn the Nursery Unit ON again to reconnect both units. To reactivate the out-of-range function, follow steps above. If you turn the **Parent Unit OFF**, the Out-of-Range function will be deactivated but the orange light will stay lit on the Nursery Unit. To reactivate this function on the Nursery Unit follow steps above.

5.3 - SOUND

TO TEST SOUND:

Ask someone to speak into the Nursery Unit or place the unit near a radio to ensure that sound is being picked up.

IMPORTANT


To avoid feedback (whistling), keep both units at least 3 meters apart.

IF YOU HEAR NO SOUND:

- Check that both units are switched on.
- Check that both units are plugged into functioning electrical outlets.
- Check the batteries in both units (replace or recharge them as necessary per unit).
- Bring the Parent Unit closer to the Nursery Unit.

TO ADJUST SOUND LEVEL:

Slide Parent Unit Volume/Power button upward or downward.

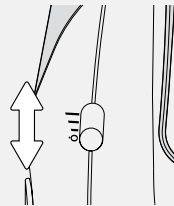


5.4 - NIGHT LIGHT

Press the **Night Light** button on the Nursery Unit to turn it **ON** or **OFF**.

NOTE:

If you press Night Light button for more than 2 seconds Out-of-Range function automatically activates.

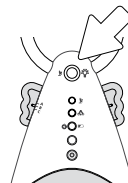

TAKING CARE OF YOUR RECHARGEABLE BATTERIES

- Ensure that you fully charge your batteries for 15 hours before their first use.
- To maintain maximum battery life it is recommended that you fully discharge your batteries periodically. Battery performance can be diminished by leaving the batteries constantly charging. If your parent unit is not in use, please turn it off.
- Inadequate battery power can impair the performance of your monitor. If you are not happy with the performance of your Parent Unit/Monitor please replace the rechargeable batteries (3 or 6 x NiMH Rechargeable Batteries).
- It is advisable to change the batteries every 6 months

FALSE ALARMS

- Verify the connection between Sensor Pad and Nursery Unit. Unplug and reconnect the cord.
- Make sure there is no bedding between the mattress and the Sensor Pad.
- If the crib has a spring or slat base, place a hardboard between the crib base and the Sensor Pad measuring at least 33 x 33 cm and 6 mm thick.
- Make sure the Sensor Pad is positioned as described under the setup requirements. Sensitivity may need to be adjusted, refer to step 5.1.

ALARM WILL NOT SOUND

- Avoid contact with the crib when the monitor is ON.
- Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside the crib.
- Turn OFF crib mobile, fans, loud music or anything else that may cause vibration.
- You may need to decrease the sensitivity, refer to step 5.1.

NO SOUND IS BEING TRANSMITTED

- Ensure that both units are switched ON.
- Check that the sound has not been turned down on the Parent Unit. Increase the volume level on the Parent Unit, refer to step 5.3.
- If there is no power to any unit check/replace the batteries, recharge the Parent Unit & check the AC Adaptors are working. Change electrical outlets if required.
- Bring the Parent Unit and Nursery Unit closer together.

POWER INDICATOR (GREEN LIGHT) DOES NOT LIGHT UP

- Check/replace batteries or recharge the Parent Unit.
- Check connections or change to another electrical outlet.
- Ensure both units are on the same channel, either A, B or C.

WEAK RECEPTION

- Bring units closer together.
- Change the position of one or both units.
- Place units on a flat surface away from baby's reach.
- Replace the batteries or recharge the Parent Unit.

STATIC, DISTORTION, INTERFERENCES (FROM OTHER MONITORS, WIRELESS PHONES, WALKIE-TALKIES, ETC...)

- Rotate the Parent Unit or move it away from source of interference.
- Check/replace batteries or recharge Parent Unit.

FEEDBACK (LOUD NOISES EMITTED FROM THE PARENT UNIT)

- Move the units farther apart (at least 3 meters).
- Decrease volume on the Parent Unit.

LIMITED WARRANTY

Angelcare® warrants to the original owner of this product that Angelcare® rechargeable, movement and sound monitor (model AC601) is free from any material and workmanship defects, for a period of one (1) year (rechargeable batteries not included) from the date of the original purchase. If this monitoring system fails to function properly when used as directed under normal conditions within the one year period, Angelcare® will either repair or replace the product, at our discretion, free of charge. Please contact your distributor.

The product must be accompanied by a proof of purchase, either a bill of sale or other proof that the monitoring system is within the warranty period. Angelcare® will bear the cost of repairing or replacing the product and shipping it back to you.

This warranty does not apply to a product which has been damaged as a result of improper maintenance, an accident, improper voltage supply or any other form of misuse. The warranty is also void if the owner repairs or modifies the product in any way. Angelcare® is not liable for any incidental or consequential damages with regard to this product. The warranty also excludes any liability other than what is stated above. No other warranty is given.

This warranty does not cover products and accessories which are not Angelcare-branded, or any problems that result from usage that is not in accordance with product instructions, failure to follow the product instructions, or problems caused by using accessories, parts or components not supplied by Angelcare.

LEGAL RIGHTS VARY FROM COUNTRY TO COUNTRY. SOME COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

WE ARE NOT IN ANY WAY RESPONSIBLE FOR ANY OUTCOME RESULTING FROM THE USE OF THIS PRODUCT.

IMPORTANT!

This Angelcare® monitor is a personal care product. Please do not return this product to the store. If you have questions or need help, please call your distributor before returning this product.

TECHNICAL SPECIFICATIONS

Range:

Open field (Open area without obstructions): up to 250 meters.

Number of Channels:

USA/Canada: 3 channels.
Europe: 3 Channels.

Power Supply:

4 AAA alkaline batteries (not included) or 100-240V AC/7.5V DC Adapter for Nursery Unit.

3 AAA rechargeable batteries (included) and 100-240V AC/7.5V DC adapter for the Parent Unit.

Caution: To reduce the risk of electric shock, do not expose this product to rain or moisture.

Danger of explosion if batteries are incorrectly replaced. The included batteries are 3 AAA NIMH rechargeable batteries manufactured by GPI INTERNATIONAL LIMITED (GN60AAAHC, 1.2V 600MAH).

They may only be replaced by 3 AAA NIMH rechargeable batteries.

Batteries:

- DO NOT mix old and new batteries.
- DO NOT mix alkaline, standard (carbon-zinc) or rechargeable (NI-CD, NIMH, etc) batteries.

Transmission:

Movement signal and sound transmission from Nursery Unit to Parent Unit is made on the 864MHz frequency. Transmission from the Nursery Unit to the Parent Unit is made on the 864 MHz frequency in Europe/Asia/Africa and on the 927 MHz frequency for North America/Ecuador/Oceania.

Hereby, Angelcare Monitors Inc, declares this baby monitor is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

The EU declaration of conformity can be obtained at: **www.angelcarebaby.com**



PROTECT THE ENVIRONMENT

This electronic equipment contains reusable material and must not be disposed with domestic waste. Please take it to a collecting point for electric and electronic equipment (ask your local authorities for details).

ErP COMPLIANCE

NORTH AMERICA

Use only Angelcare® brand power supply - model number: T07505U002

UK- MALTA- CYPRUS - SINGAPORE - HK- UAE

Use only Angelcare® Brand power supply - model number: T07505B001

EUROPEAN UNION - SOUTH AFRICA - BRAZIL - RUSSIA

Use only Angelcare® Brand power supply - model number: T07505V001

GERMANY

Use only Angelcare® Brand power supply - model number: T07505F002

AUSTRALIA

Use only Angelcare® Brand power supply - model number: T07505S002 (MEPS)

This Angelcare Baby Monitor is a Personal Care Product. **Please DO NOT return this to the store**, but contact us initially for advice and assistance:

Angelcare Helpline on **0345 009 1789**

Website: www.angelcare.co.uk

Email: helpline@babyhelpline.co.uk

NOTE: Helpline is open Monday to Friday, 10am to 4pm, excluding bank holidays.

